


PRE-WIRED ECG ELECTRODE

FOR USE WITH:


Product No	Reference No
6502875V	A10055-SRT
6502874V	A10077-SRT
6502876V	A10058-SRT

DURATION OF USE

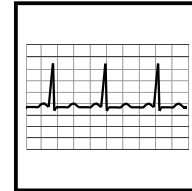


● < 120 hours

INSTRUCTIONS FOR USE

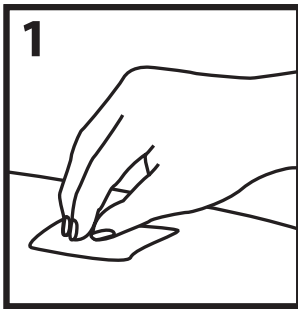


• Prep

- A. Clean & Dry Skin
- B. Moisture inhibits adhesion
- C. Do not use alcohol

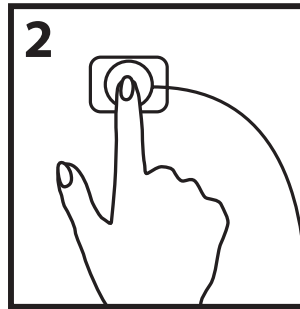

• Apply

- A. Remove electrodes
- B. Warm for 45 seconds
- C. Hold in place for 60 seconds

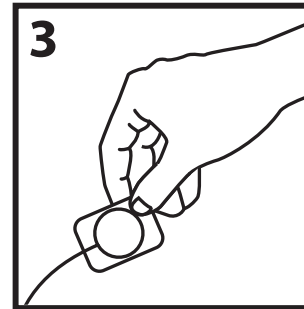

• Remove

- A. Soak electrode with water
- B. Gently peel away
- C. Soap is also usable

CAUTIONS

- Federal law restricts this device to sale by or on the order of a physician.
- ECG electrodes should only be used by or in consultation with a health care provider familiar with their proper placement and use;
- ECG electrodes may damage the skin if removed carelessly;
- ECG electrodes should be applied only to intact, clean skin (e.g., not over open wounds, lesions, infected, or inflamed areas); and
- ECG electrodes should be properly disposed of after use
- Minor skin irritation can occur with electrode use. If skin irritation occurs, please contact your prescribing physician or health care professional.

PRECAUTIONS

- During surgical procedures, electrodes should be placed as far as possible from any electro-surgical area to minimize unwanted RF current flow.
- Self-adhesive electrodes should be replaced if they no longer stick firmly to the skin.

VML172 (04/2021)



HS.NisshaMedical.com/IFU



Not made with natural rubber latex

